DE

**Installationsanleitung
FRIATRACE Version 5.3**

EN

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Contents

	Page
1. Summary of FRIATRACE	10
2. FRIATRACE Installation	11
3. Installation program	11
4. Installation under Microsoft® Windows 2000, Windows X, Windows 7 (32bit) or Windows 8.1	12
5. Installation under Microsoft® Windows 7 (64bit)	13
6. Update of these installation instructions	14
7. FRIATRACE Software Licence	14

1. Summary of FRIATRACE

FRIATRACE is a very powerful tool for the administration of your fusion records. FRIATRACE has its own data bank with comprehensive search, sort and administration functions. FRIATRACE is available for Microsoft® Windows 2000, Windows XP, Windows 7 and Windows 8.1.

With FRIATRACE you can:

- Read out fusion records from all FRIAMAT® types (with documentation).
- Transfer fusion records from the Memory-Box, Memory Card und Memory Stick.
- Transfer records from the earlier version of FRIATOOL IV.
- Delete the content of the Memory-Box, Memory Card und Memory Stick and check the status of the Memory-Box battery.
- The data from the fusion records can be:
 - displayed as tables, processed and printed.
 - displayed as continuous text and printed out.
 - itaken into an integral data bank, selected, processed, relocated and printed.
 - saved in a wide variety of file formats and exported (dBase III, Text file, Microsoft® Access and Microsoft® Excel).
- print out bar code labels for commission numbers.

Hardware requirements for Microsoft® Windows 2000 / Windows XP / Windows 7 / Windows 8.1:

- PC with a Pentium 90 Processor or higher
- 16 MB RAM (32 MB recommended)
- Memory space required on the hard disc (15 MB)
- Mouse or compatible pointer device recommended
- CD-ROM drive
- Parallel 25 pin interface (printer connection)

2. FRIATRACE Installation

FRIATRACE is installed with the aid of an installation program. This installation program will guide you through the installation step by step and carry out all necessary work (copying the files, creating directories etc.). Before you start the FRIATRACE-installation program you should ensure that all other applications on your computer have been terminated.

FRIATRACE is available on CD.

An Update of FRIATOOL I, II and III up to FRIATRACE is not possible. Please note that for installation under Microsoft® Windows 2000, Windows XP, Windows 7 and Windows 8.1 you must be signed on as the system administrator!

Select "START Settings Control-Panel". Start "Add/Remove Programs" with a double click. You will go into the dialogue "Add/Remove Programs Properties". In the "Install /Uninstall" register choose the "Install" button.

The dialogue "Install program from CD" will appear. You will be requested to place the CD into the appropriate drive. Follow this and subsequent instructions to start the installation program.

3. Installation program

As already mentioned the installation program will guide you step by step through the installation of FRIATRACE. You should be prepared to give the following details:

- State the program directory

The installation program needs to know in which directory FRIATRACE is to be installed. It is suggested that C:\FRIATRACE be used.

- Other directories

FRIATRACE can file records in a data bank. You can specify the directory in which the data bank is to be saved. It is suggested that program directory \DATA be used.

Furthermore you can state a directory in which the original records and the processed records are to be saved. In each case you will be given program directory \PROTOKOL as default.

To print out from the data bank you use so-called layouts. Enter the directory in which these layouts are to be saved. Program directory \LAYOUT will be given as default.

In some circumstances some files will be overwritten during installation. The installation program will offer you the option of saving these files automatically during installation. Enter the directory in which you would like to store the saved files. Program directory \BACKUP will be given as default.

- User information

You will have the option to enter the Name and Address of your company.

- Serial number/ Licence number

You must enter the Serial Number which came with the software. Without a valid Serial Number FRIATRACE cannot be installed.

At any point you can return to the previous stage in the installation process or break off installation completely.

At the end of installation Windows will have to be restarted. You will then be requested to restart your system. Until Windows has been restarted, you will not be able to work with FRIATRACE.

4. Installation under Microsoft® Windows 2000, Windows X, Windows 7 (32bit) or Windows 8.1

Please note:

After the installation under Microsoft® Windows 2000, Windows XP, Windows 7 (32bit) or Windows 8.1 the authorisation for some codes must be set in the registry. In the program directory for FRIATRACE you will find the file README.WRI which contains more detailed information.

After installation has been completed (and Windows has been restarted) FRIATRACE will be available for use.

5. Installation under Microsoft® Windows 7 (64bit)

Please note:

Depending on the system configuration it can appear that during installation of FRIATRACE under Microsoft® Windows 7 (64bit) the system constantly reboots even before the login dialog appears. In this case please proceed as follows:

1. Insert FRIATRACE CD and start the setup with administrator rights. Then restart.
2. The computer starts and Microsoft® Windows 7 makes a restart even before the login window appears.
3. Now right after this restart open the start menu of Windows with the F8 key.
4. Select the option "Last known good configuration" in the menu.
Now, Windows 7 boots normally with user rights.

To run FRIATRACE under restricted rights, the following changes in the registry of Windows 7 (64bit) are required:

1. To access the Windows registry editor, press the Windows-key and R-key simultaneously. Enter the command "regedit" and perform with the Enter-key.
2. Now the permissions for the registry key "FRIATEC AG " must be set in the path:"HKEY_CLASSES_ROOT\VirtualStore\MACHINE\SOFTWARE\Wow6432Node\", as well as for the Registry key "Jet", who is "HKEY_CLASSES_ROOT\VirtualStore\MACHINE\SOFTWARE\Wow6432Node\Microsoft \".
3. As a final step the access rights for the FRIATRACE- Directory (default: C:\FRIATRACE) need to are set, which is located in the Root directory of the system disk.

Hotline: ++49 621/486-1836
Fax: ++49 621/486-2030
Email: info-friatools@friatec.de
Internet <http://www.friatec.de>

6. Update of these installation instructions

These technical statements are regularly checked for their up-to-dateness. The date of the last revision is stated on the document. For an updated version of the installation instructions, please visit our website www.friatools.com on the Internet. You will find the “Download” page on the navigation bar. This page contains our updated instructions as pdf documents. We would also be pleased to mail them to you on request.

7. FRIATRACE Software Licence

The granting of a licence. This licence agreement gives the LICENSEE the right to use a copy of the software program FRIATRACE on a single computer. The software is „in use“ on a computer when it is loaded into the working memory (i.e. RAM) or saved in a permanent memory (e.g. a hard disc, a CD-ROM or some other storage device). If the anticipated number of users of the SOFTWARE might exceed the number of licences granted, the LICENSEE must take appropriate measures to ensure that the number of persons using the SOFTWARE does not exceed the number of licences. FRIATEC AG reserves all rights, which are not expressly granted to the LICENSEE in this contract.

Copyright. The SOFTWARE is the property of FRIATEC AG or its suppliers and is protected by the applicable copyright laws and international treaties. Therefore the LICENSEE must treat the SOFTWARE like any other work protected by copyright. The LICENSEE may either (a) make a copy of the SOFTWARE exclusively for security or archiving purposes or (b) transfer the SOFTWARE onto a single hard disc, as long as the LICENSEE retains the original exclusively for security or archiving purposes.

Other limitations. The LICENSEE may not redevelop, decompile, or disassemble the SOFTWARE where this is not conclusively stipulated by statutory regulations. The LICENSEE may transfer the SOFTWARE to a third party provided that after the transfer the LICENSEE retains neither a copy of the SOFTWARE, including all updates, which the LICENSEE may have received, nor any written documentation of any kind that was delivered with the SOFTWARE.

Operating system options (CD-ROM Media). If the SOFTWARE is marketed on a CD-ROM with several versions for compatibility with different operating systems the LICENSEE is permitted by his licence to utilise only one SOFTWARE version which is suited to the LICENSEE'S operating system.

Warranty. FRIATEC AG warrants that for a period of 6 months from the date of receipt, (a) the SOFTWARE will perform substantially in accordance with the written documentation enclosed, and (b) the data carrier on which the SOFTWARE is stored will be free from defects in material and workmanship under normal use and service.

In no event shall FRIATEC AG or its suppliers be liable for any damages whatsoever (including, without limitations, damages or loss of business profits, business interruption, loss of business information/data, or any other pecuniary loss) arising out of the use of or inability to use this SOFTWARE. This also applies if FRIATEC AG has been informed about the possibility of such damages. In any case the liability of FRIATEC AG is restricted to the amount of the actual purchase price. This exclusion does not apply to damage caused intentionally or due to gross negligence on the part of FRIATEC AG. Claims based on inalienable legal regulations on product liability are likewise unaffected.

FRIATEC Aktiengesellschaft
Division Technische Kunststoffe
Postfach 71 02 61 · D-68222 Mannheim
Telefon +49 621 486-1533
Telefax +49 621 486-2030
info-friatools@friatec.de
www.friatools.de

FRIATEC Aktiengesellschaft
Technical Plastics Division
P.O.B. 71 02 61 · D-68222 Mannheim
Phone +49 621 486-1533
Fax +49 621 486-2030
info-friatools@friatec.de
www.friatools.com